Silikon Nasenbrille „Everest“

Produkt Information:

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Silicone Nasal Cannula „Everest“

Product Information:

- Product Reference Code – 1200 / 1800 straight prong, 1200 / 1800 curved prong
- Description - 3.5mm silicone nasal cannula
- Tubing length - 1200mm / 1800mm
- Material - 70 shore translucent silicone
- Non-sterile
- Autoclavable
- For use with medical grade oxygen only
- Maximum flow rate - 5-ltr/min
- **Latex Free**

Caution:

- Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death.

DO NOT SMOKE WHILST USING OXYGEN!

- DO NOT USE near open flames or very hot ignition sources
- Do not store in direct sunlight.
- Do not store in high temperature locations.
- Store in dry conditions.
- Do not use if this packaging has been opened or damaged.

Directions:

- Attach oxygen tubing to the oxygen flow meter.
- Set the oxygen flow as prescribed and place the nasal prongs **briefly underwater** to see that oxygen is flowing properly.
- **If there are no bubbles appearing in the water, do not use the cannula.**
- Insert nasal tips into the nostrils.
- Pass the 2 silicone tubes over the ears and under the chin. Use the toggle to adjust the cannula so that it fits snugly.

Hygiene: Product suitable for autoclaving. In absence of this facility follow the instructions below

- To disinfect, place for 20 minutes in boiling distilled water at least every 2 weeks or more often.
- Dry by flowing oxygen through the cannula.
- Do not use the cannula if water is still in the tubing.
- Possible to clean in vaporizer

To remove surface dirt, wipe the cannula with a damp cloth containing a very weak solution of detergent

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Medical Instruments Corp. GmbH