RESMED

DC/DC Converter 24V/90W

User Guide

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • 简体中文 • 繁體中文 • 日本語

Using the converter via a cigarette lighter

Refer to the illustration at the beginning.

1. Before connecting the converter, ensure the switch **(A)** is off.
 2. Position the converter on a level surface, with the ResMed logo facing up.
 3. Plug the connector end **(B)** of the converter into the DC input socket **(C)** of the S9 device.
 4. Remove the cigarette lighter from the vehicle socket.
 5. Plug the cigarette lighter adaptor plug **(D)** on the converter into the vehicle cigarette lighter socket.
 6. The light **(E)** on the converter will glow green when the connection is made and the switch is on. The S9 device should then power on.
- The light will flash continuously if the connection is not made, or if the device is in standby mode or power save mode when the connection is made. If the battery is low, the light will turn amber. If the light does not operate, check that the switch on the cigarette lighter adaptor plug is on.

Notes:

- Some vehicles require the ignition key to be turned to the "accessories" position to provide power to the cigarette lighter socket.
- If you connect the S9 device to a powered converter, the converter may power down. To resume power, turn the converter off and on again via the switch on the cigarette lighter adaptor plug.
- If the battery AND car, boat or other vehicle are not in good condition, there may not be enough reserve left in the battery to start the vehicle after use.
- ResMed does not recommend frequent use of a standard vehicle battery to power the S9 device. A standard vehicle battery is not optimized for frequent heavy discharges. Contact ResMed Technical Services for further information and recommendations.
- The S9 system with the H5i humidifier attached uses considerably more power than the S9 system alone. Using the H5i will drain the battery more quickly.
- If the converter cuts power to the S9 device when starting your car, boat or other vehicle, disconnect the converter from the vehicle, wait a few seconds, and then reconnect it.
- Do not leave the converter connected to the power source when the S9 device is not in use, as this may discharge the battery over time.
- You must have an electrical system in good condition. Have it checked by an auto electrician if in doubt.
- The DC/DC converter can also be used with battery clips. For more information, see the battery guide available on www.resmed.com on the Products page under Service & Support.
- Contact ResMed or a representative if you need help.

English

The DC/DC Converter 24V/90W allows you to operate an S9™ device including an H5i™ and ClimateLine™ from a 12V or 24V DC power source in a car, boat, or other vehicle equipped with a suitable battery. Consult your equipment supplier for more information about this accessory.

⚠ WARNINGS

- Read instructions before use.
- Only use the converter when connecting to a 12V or 24V DC supply.
- Do not connect the S9 device to a 12V or 24V DC power source while the power source is being charged from AC mains.
- Do not attempt to modify the converter. There are no user-serviceable parts inside the converter.

⚠ CAUTIONS

- The converter must only be used with S9 devices to ensure proper performance of the device.
- The converter is intended for use on 12V or 24V negative ground systems (where the negative terminal of the battery is grounded). Using it on a 12V or 24V positive ground system may damage the S9 device and the vehicle's electrical system. If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.
- Make sure the converter is not covered during use to avoid overheating the device.
- The converter is not intended for use on an aircraft.

Technical specifications

Length (connector to connector): 2.3 m

Fuse rating: F 20A L, 250 V (fast blow, low breaking capacity)

Input: 12 V/24 V $\overline{\text{---}}$ 13A/6.5A

Output: 24 V $\overline{\text{---}}$ 3.75A

Typical power consumption: 70 W

Maximum power consumption: 110 W

Environmental conditions:

Operating temperature: +5°C to +35°C;

Operating humidity: 10 to 95% non-condensing;

Operating altitude: Sea level to 2,591 m; air pressure range 1013 hPa to 738 hPa;

Storage and transport temperature: -20°C to +60°C;

Storage and transport humidity: 10 to 95% non-condensing.

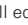
Electromagnetic compatibility:

Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial and light industry environments.

Ensure that wireless communications devices (such as mobile phones, wireless home network devices, cordless telephones and their base stations, and walkie-talkies) are kept at least 3.2 ft (1 m) away from the device.

Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on www.resmed.com, on the Products page under Service and Support.

Symbols which may appear on the device

 Caution;  Class II equipment;  IP21 Protection against insertion of fingers and against vertically dripping water;

 Direct Current;  Manufacturer;  **EC/REP** European Authorized Representative;  **RoHS** European RoHS;  **SN** Serial

number;  China pollution control;  Fuse;

 **Environmental Information** The DC/DC converter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment. If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Limited warranty

ResMed warrants that your ResMed DC/DC Converter 24V/90W shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. **EC REP** ResMed (UK) Ltd 96 Jubilee Ave Milton Park Abingdon Oxfordshire OX14 4RW UK. See www.resmed.com for other ResMed locations worldwide.

S9, H5i and ClimateLine are trademarks of ResMed Ltd and S9 and ClimateLine are registered in U.S. Patent and Trademark Office.

©2013 ResMed Ltd. 368918/1 2013-06