# ResMed

## Addendum

English | Български | Hrvatski | Čeština | Dansk | Nederlands | Eesti | Français | Português |  
Español | Suomi | Deutsch | ΕΛΛΗΝΙΚΑ | Magyar | Íslenska | Italiano | Latviski | Lietuvių k. | Norsk |  
Polski | Română | Русский | Српски | Slovenčina | Slovenščina | Svenska | Türkçe |  
اردو | فارسی | עברית | العربية

### ENGLISH

#### ATTENTION

The information in this addendum provides important updates to the Contraindications and Warnings found in the mask User Guide.

#### Contraindication

Masks with magnetic components are contraindicated for use by patients where they, or anyone in close physical contact while using the mask, have the following:

- Active medical implants that interact with magnets (i.e., pacemakers, implantable cardioverter defibrillators (ICD), neurostimulators, cerebrospinal fluid (CSF) shunts, insulin/infusion pumps)
- Metallic implants/objects containing ferromagnetic material (i.e., aneurysm clips/flow disruption devices, embolic coils, stents, valves, electrodes, implants to restore hearing or balance with implanted magnets, ocular implants, metallic splinters in the eye)

#### WARNING

Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. This warning applies to you or anyone in close physical contact with your mask. The magnets are in the frame and lower headgear clips, with a magnetic field strength of up to 400mT. When worn, they connect to secure the mask but may inadvertently detach while asleep.

Implants/medical devices, including those listed within contraindications, may be adversely affected if they change function under external magnetic fields or contain ferromagnetic materials that attract/repel to magnetic fields (some metallic implants, e.g., contact lenses with metal, dental implants, metallic cranial plates, screws, burr hole covers, and bone substitute devices). Consult your physician and manufacturer of your implant / other medical device for information on the potential adverse effects of magnetic fields.

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© 2023 ResMed 638359/2 2023-08
**ResMed Pty Ltd**